

# iNurse®

Længere amning - sundere børn



## Product Information

### HOW TO USE:


iNurse® is to be used when breastfeeding mums experience pain or uncomfortableness.

In Denmark 8 out of 10 mums experience this. It promotes healing of the breast and nipple and protects from friction and irritation from clothing and underwear.

It is to be used between breastfeeds, under nursing bra.

It results in quicker healing for the nipple/breast due to the air circulation it provides.

Excess milk and fluids will be caught by the additional bra pad, therefore the risk of fungus and bacteria is greatly reduced. It is also beneficial for non-bacterial breast infections due to the allowance of the breast to dry without being exposed.

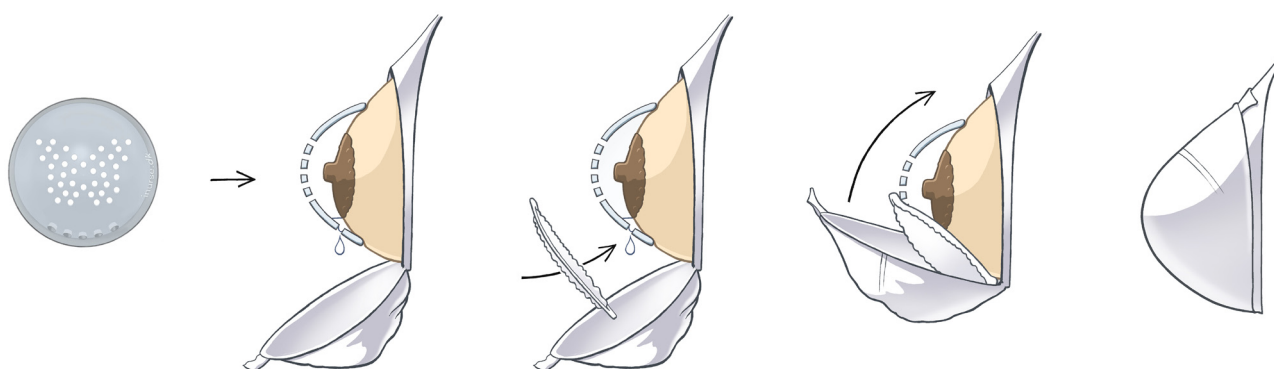

The vision behind the development of iNurse: Help women to breastfeed longer with reduced pain.

January 2017 - 1 of 2 pages

# iNurse®

Længere amning - sundere børn



## Product Information (Continued..)

### THE SHAPE:

A little soft shell – developed with the help of one of the country's leading prosthetists – with air holes in the shape of a butterfly that secures excellent ventilation, and therefore fastens the healing process. The 5 outlet holes at the bottom ensures the excess milk to be quickly and efficiently guided to the bra pad.

### MATERIALS:

Medically approved plastic, that allows contact with skin 24/7.

Phthalate and silicone free, contains no harmful contents and is perfume free.

Meets the Pharmacy sector requirements.

Can be boiled or washed without losing function or shape.

### IMPORTANT INFORMATION FOR MUMS:

Nursing bra must NOT be too tight in order to allow room for both the iNurse® and pad.

### CLEANING, WASHING INSTRUCTIONS:

Wash the same as feeding bottles, without the loss of shape or function.